



Item #: MI391

extriCARE® 3000

Negative Pressure Wound Therapy System

Distributor Sell Sheet



extriCARE® 3000

Negative Pressure Wound Therapy System

Mindfully designed with the purpose of improving performance and intuitive usability, the **extriCARE® 3000** pump is versatile, lightweight, and quiet – which is ideal for a wide range of wound applications and environments. A vibrant LCD screen provides a simple user interface and clear status indications. Using proprietary algorithms, the **extriCARE® 3000** pump has intelligent pressure adjustments, providing gradual and gentle suction for patient comfort, and the ability to store exudate collection time and usage data – a first in the extriCARE family. The **extriCARE® 3000** pump has multiple canister sizes (100cc or 400cc) for added convenience.

Features and Benefits:

- ✔ Intelligent pressure sensing and adjustment for gradual and gentle suction
- ✔ Tracks data usage and exudate level timeline
- ✔ Usage log with date and time stamp
- ✔ Powerful yet versatile, lightweight, and quiet - intended for a wide range of wound applications and environments
- ✔ User-friendly operation and interface
- ✔ Long-lasting battery life
- ✔ Wide pressure ranges from 20-140mmHg
- ✔ Continuous and Intermittent Modes


Indication for Use:

The **extriCARE® 3000 Negative Pressure Wound Therapy System** is intended to generate negative pressure or suction to remove wound exudates, infectious material, and tissue debris from the wound bed.


It is indicated for the following wound types:

- Chronic
- Acute
- Traumatic
- Subacute and dehisced wounds
- Partial-thickness burns
- Ulcers (such as diabetic or pressure)
- Flaps and grafts


User Interface:

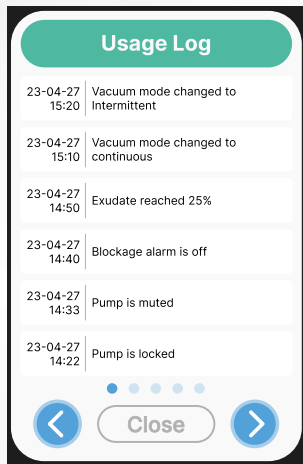

On-Screen Usage Log

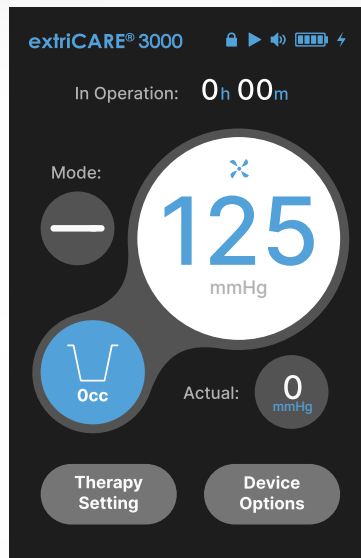

Main therapy interface

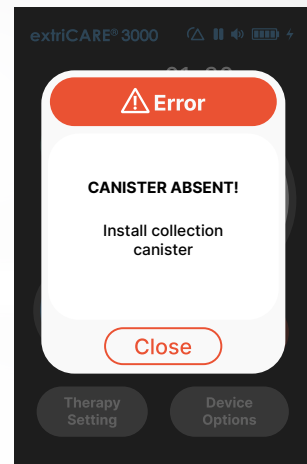



Easy-to-read error message

Simple but effective. The extriCARE 3000 features a prominent 3.5" high-resolution touchscreen that enables easy identification of the therapy settings and operating status. The resistive touchscreen works well with gloves, bare hands and through clear panel on the carrying case. Adjustment of pressure and changing of vacuum mode is done by a simple press. Any error message is displayed in large font with suggestion to mitigate the error, along with a red LED indicator light. There is no guesswork or need to refer back to the Manual to find out what error is occurring.


Software Intelligence

Usage Log, data tracking and canister monitoring

The extriCARE 3000 is equipped with proprietary technology that allow it to track exudate level and data usage that can be exported to a computer. Data can be then analyzed with a data dashboard. It also features a comprehensive on screen usage log with time and date stamp as a reference tool when needed.

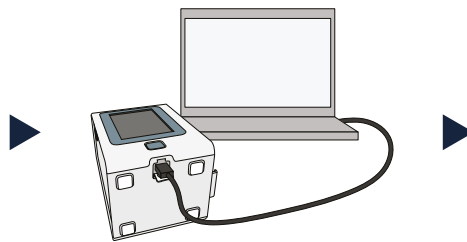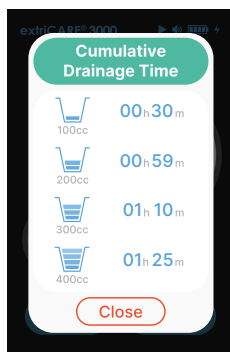

Downloading

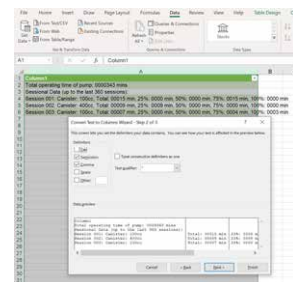

Output file

Accessories:



100cc collection canister
(ref# : EC3000-100cc)



400cc collection canister
(ref# : EC3000-400cc)



Carrying case for
100cc collection canister



Carrying case for
400cc collection canister



Charger

Device Specifications:

Product Code:	EC3000 (MI391)
Weight:	1.37 lbs (0.62 kg)
Battery Type:	Lithium (rechargeable)
Vacuum Range:	20-140 mmHg
Vacuum Modes:	Continuous or intermittent
Charging Time:	<4.5 hours
Dimensions:	Length: 4.0" (10.2cm) Depth: 3.1" (7.9cm) Height: 5.0" (12.7cm)
Inputs:	100-240 V 50/60 Hz 0.8 A

Order Information:

Order Code	Product Description	Packaging
EC3000 (MI391)	extriCARE 3000 NPWT unit	4 units per shipper
EC3000-100C	100cc Canister for extriCARE 3000 unit	80 pieces per shipper
EC3000-400C	400cc Canister for extriCARE 3000 unit	20 pieces per shipper
EC3000-CA100	Carrying case for EC3000 unit w/ 100cc canister	50 pieces per shipper
EC3000-CA400	Carrying case for EC3000 unit w/ 400cc canister	50 pieces per shipper
EC3000-CHGR	Charger for extriCARE 3000 unit	20 pieces per shipper
EC3000-CLAMP	Bed/Pole clamp for extriCARE 3000 unit	5 pieces per shipper



Alleva Medical Device

Unit B, 1050 W Central Avenue, Brea, CA, USA 92821
Tel: 714-987-3603 info@allevamedical.com

A subsidiary of Alleva Medical Ltd

Suite M-Q, 12th Floor, Kings Wing Plaza 2, 1 On Kwan Street, Shatin, Hong Kong
Tel: +852 3166 7200 www.allevamedical.com

Distributed in the USA exclusively by

ExtricareUSA LLC

Suite 205, 2460 General Armistead Avenue, Norristown, PA 19403
Tel: 877-312-NPWT www.extricareusa.com

All products are manufactured under US FDA, ISO 13485, and Japan PMDA registered facility of Alleva Medical Ltd.

Alleva Medical®, extriCARE®, Vide®, Maxson® are Trademarks and/or Registered Trademarks of Alleva Medical Ltd., its subsidiaries, and/or affiliates in the United States, and China and other countries. Copyright 2023.

All dimensions, weights, colours and contents may vary.



www.extricareusa.com



www.allevamedical.com